
# LELO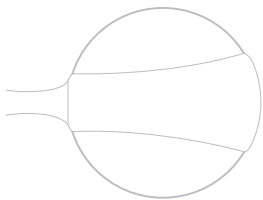

**LUNA BEAD USER MANUAL**

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The vagina is a fascinating work of art. Given proper attention and well looked after, it can display vice-like strength and provide exquisite pleasure. Use it wisely, often and with care.

Congratulations on acquiring the LUNA pleasure bead system. Before getting more closely acquainted with your LUNA beads, please take a moment to look through this user manual.

## INTRODUCTION


As with any muscle group, the vaginal and pelvic floor muscles may become weakened and lose their elasticity if not exercised on a regular basis. Factors leading to a slackening of muscle strength range from frequent heavy lifting to pregnancy, childbirth and the hormonal changes that accompany aging.

Performing daily Kegel exercises to exercise the pelvic floor and vagina helps increase muscular strength and provides a sense of control over the body, boosting sexual confidence and general health. And of course, it enhances erotic sensation and lays the foundation for intense orgasms.

**The LUNA bead system is a combined pleasure / exercise system for the circum vaginal and pelvic floor muscles. By marrying incremental resistance with a movement-induced subtle kinetic vibe, LUNA beads facilitate more efficient and pleasurable Kegel exercising.**

## LUNA BEAD OVERVIEW

The LUNA pleasure bead system consists of a silicone composition girdle and four weighted beads. The weight of each bead is identified by its colour: two petal pink 28-gram beads (one foundation bead with retraction leash and one crown bead) and two powder blue 37-gram beads (one foundation bead with retraction leash and one crown bead). The weight range within the basic system is 28 to 74 grams.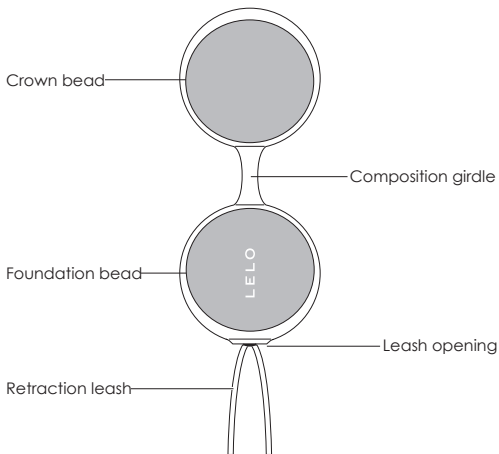

## COMPOSITION

The foundation beads can be used individually or combined with a crown bead using the composition girdle.

- 1. Thread the retraction leash, on the foundation bead of your choice, through the leash opening in the foundation bead nook on the composition girdle.**
- 2. Push the foundation bead into place, making sure the edges of the nook slot firmly into place and trace the contours of the girdle guide imprint on the foundation bead.**
- 3. Select a crown bead. Push the bead into place, making sure the edges of the crown nook slot firmly into place and trace the contours of the girdle guide imprint on the crown bead.**

Depending on your level of vaginal and pelvic floor fitness, you may wish to start out with a single foundation bead and gradually progress to compositions involving two beads and more weight. The most important thing is that the composition you opt for feels comfortable.

## **INSERTING LUNA BEADS**


Inserting LUNA beads is not dissimilar to inserting a tampon. As LUNA beads have a wider girth, however, some additional tricks apply.

- 1. To facilitate easy insertion, apply a small amount of lubricant on the leading edge of the bead(s) you intend to use. Avoid using too much, as this will make the bead(s) slippery and difficult to hold in place.**
- 2. Lie on your back and introduce the leading end of the bead(s) into your vagina with a firm but gentle push. If you experience difficulties, raise your pelvis so that gravity can assist you. Some women find that a standing position, with one leg raised, works better for them.**
- 3. If you are using two beads and the composition girdle, insert the crown bead first. Due to the gap between the two beads, once the crown bead is inserted the vaginal muscles may try to push it out before you can insert the foundation bead. Take a deep breath and relax.**

## POSITIONING

The bead(s) should be positioned so that the lower edge of the foundation bead is at least 2 cm inside you. The most important thing is that the foundation bead rests above your pelvic floor muscle. The exact distance from the vaginal opening to the pelvic floor muscle differs from woman to woman, but the bead(s) should be positioned comparably to where a tampon would normally be positioned.

Always allow the retraction leash to protrude slightly from the vagina so as to facilitate removal of the bead(s).

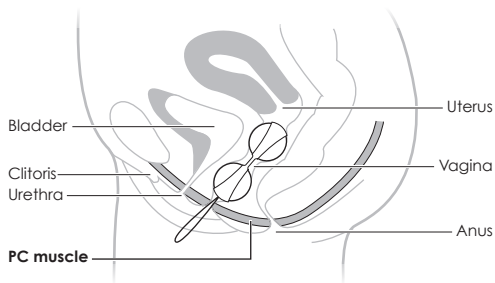## USING LUNA BEADS

If you intend to use the LUNA bead system primarily for exercise purposes, it is important that you incorporate it into your daily routine and adhere to a fixed program.

Depending on your level of vaginal and pelvic floor fitness, it may take up to 12 weeks of daily 30-minute Kegel exercising using LUNA beads to achieve noticeable results.

At the outset of using LUNA beads as part of your Kegel routine, you may feel slightly uncomfortable, as your body will be very aware of the beads. You may also find that beads tend to slip out after only a few minutes. If this happens, do not worry, and persist with the program. As your muscles become stronger, they will automatically hold the beads in place.

When you walk, run, swim or otherwise move about, the motion causes the inner balls to move, creating a subtle kinetic vibe stimulating the vaginal muscles.

Even when you are standing still, gravity will instigate spontaneous pelvic floor muscle contractions as your muscles work to prevent the beads from slipping out.

## **CLEANING, STORAGE AND SAFETY**

The lifespan of your LUNA beads will be greatly increased with proper maintenance and care.

**LELO recommends that you NOT use a silicone-based lubricant with your LUNA beads, as this could make the silicone composition girdle permanently tacky. A water-based lubricant is the safest option. If you wish to use a silicone-based lubricant, perform a patch-test first. Never use massage oil or hand cream as lubricants.**

**Always disassemble and clean each part of the LUNA bead system thoroughly before and after each use. Clean using warm water and antibacterial soap, rinse with clean hot water and pat dry with a lint-free cloth or towel. Never use cleansers containing alcohol, petrol or acetone.**

**Avoid leaving your LUNA pleasure bead system in direct sunlight and NEVER expose it to extreme heat.**

**Store your LUNA pleasure bead system in a dust-free place and keep it away from toys of other materials.**

**Do NOT use the LUNA pleasure bead system during pregnancy.**

## SPECIFICATIONS

MATERIALS	ABS / body safe silicone
FINISH	glossy / pearl
GIRDLE SIZE	89 x 35 x 16 mm
BEAD SIZE	36 mm diameter
WEIGHTS	2 x 28 grams (petal pink)
	2 x 37 grams (powder blue)

**Disclaimer:** Users of this pleasure object use it at their own risk. Neither LELO nor its retailers assume any responsibility or liability for the use of this pleasure object.

**Model may be changed for improvements without notice.**

© 2009 LELOi AB. ALL RIGHTS RESERVED.  
DESIGNED AND DEVELOPED BY LELO SWEDEN.  
ASSEMBLED BY LELO PRC.

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© 2009 LELOi AB. ALL RIGHTS RESERVED. **LELO** IS AN INTERNATIONALLY REGISTERED TRADEMARK OF LELOi AB. REGISTERED THROUGH SUBSIDIARIES IN THE U.S.A AND OTHER COUNTRIES.